

ToothMasters Bite Splint Information

Purpose: The purpose of splint therapy is to help your lower jaw function more properly. Since the treatment you are to receive is mainly orthopedic in nature, we have fabricated an orthopedic appliance to help reposition your lower jaw. This appliance will help to relax any of your jaw muscles which are in spasm and to reduce any muscle pain. There are developmental defects, peculiar oral habits, naturally occurring malocclusion (poor bite), psychologic stress, clenching or bruxing of teeth, and other problems.

Rationale: You have received an acrylic bite splint (occlusal splint). This treatment has been used for many years to keep the teeth from contacting while you sleep and to allow the lower jaw to return to a comfortable hinge position without interference and guidance from the teeth. It is essential that you wear your night guard every night and you may also wear it during the day if you find yourself clenching or grinding your teeth. When it is not in your mouth, make sure your splint is placed in water to avoid warping.

Cleaning the splint: Food will accumulate under the splint. After brushing and flossing your teeth very thoroughly, brush and rinse the inside and outside of the splint and return it to your mouth. Dental decay can be stimulated if you are not careful about the cleanliness of your mouth and splint. Using a dental soak cleanser on a monthly basis will help keep the splint fresh.